

FotoSan 630 LAD pen Guidelines



F o t o S a n

T r e a t m e n t g u i d e l i n e s

Tips and advice

Practical FotoSan treatment

FotoSan is used in the treatment of infections in the oral cavity in cases where:

1. You have access to and can apply the photosensitizer (FotoSan Agent) in contact with patogene microorganisms.
2. The light can activate the photosensitizer (FotoSan Agent).

Three different viscosities of FotoSan Agent

FotoSan Agent is available in 3 different viscosities with a concentration of 0.1 mg/ml of the active ingredient Toluidine Blue. Choose a viscosity of the FotoSan Agent which is best suitable for the actual treatment.

Agent LOW - suitable for application in root canals.

Agent MEDIUM – suitable for treating gingivitis, application in periodontal pockets or Cavities with profound carries.

Agent HIGH - suitable for application in wide periodontal pockets, peri-implantitis, pericoronitis or periodontal surgery.

Five different tips

There are 5 different types of tips included in the FotoSan kit:

1. BLUNT 4 mm tip for surface treatment.
2. BLUNT 8 mm tip for surface treatment.
3. ENDO tip for root canal treatment.
4. PERIO Long tip for periodontal pockets and furcation treatment.
5. Perio short for periodontal pockets and furcation treatment.

All tips are for single use only!

Two different FotoSan Agent packaging

You can order FotoSan Agent in:

1. Vials of 6 ml
2. Syringes of 1, 2 ml

Please find the reference number at back page.

Important information

General treatment principles

- *Longer light exposure destroys more bacteria.*
- *FotoSan Agent is also available in prefilled syringes – ready for use.*
- *The time settings in this guide are average recommendations.*
- *Acute infections or acute exacerbations in chronic conditions may benefit from treatment with double exposure time and/or repeating the treatment.*
- *The FotoSan Agent may be colorless prior to exposure. When exposed to Oxygen and activated it will become blue in color.*
- *Shake FotoSan Agent before use.*
- *The FotoSan Agent stains textiles if spilled.*
- *If the light gets too hot, we recommend you to change the cover.*
- *All tips and cover are for single use only!*

Accessories

Item No.	Description
100501	50 x adjustable disposable BLUNT tips Ø4 mm
100502	50 x adjustable disposable BLUNT tips Ø8 mm
100503	50 x adjustable disposable PERIO LONG tips
100504	50 x adjustable disposable PERIO SHORT tips
100505	50 x adjustable disposable ENDO tips
13601	6 ml vial FotoSan Agent LOW
13606	6 x 6 ml vial FotoSan Agent LOW
13701	6 ml vial FotoSan Agent MEDIUM
13706	6 x 6 ml vial FotoSan Agent MEDIUM
13801	6 ml vial FotoSan Agent HIGH
13806	6 x 6 ml vial FotoSan Agent HIGH
13850	6 x 6 ml vial FotoSan Agent, Assorted: 2 x HIGH, MEDIUM, LOW
13603	5 x 1,2 ml syringe FotoSan Agent LOW
13703	5 x 1,2 ml syringe FotoSan Agent MEDIUM
13803	5 x 1,2 ml syringe FotoSan Agent HIGH
13855	5 x 1,2 ml syringe FotoSan Agent, Assorted: 2 x HIGH, MEDIUM, 1 x LOW
13900	Protection glasses
13901	Flip-on protection glasses
315118	100 Appli-Vac bendable tips 18G, 18mm, pink
315119	100 Appli-Vac bendable tips 19G, 18mm, brown
315120	100 Appli-Vac bendable tips 20G, 18mm, yellow
315122	100 x Appli-Vac blunt needle 22G, ø 0,7x18mm, black
315125	100 x Appli-Vac blunt needle 25G, ø 0,5x18mm, blue

Manufactured by:

CMS Dental
 Njalsgade 21 G
 2300 Copenhagen S
 Denmark
www.cmsdental.com info@cmsdental.dk
 Tel.: +45 32 57 30 00
 Fax: +45 32 57 10 23

